lorelli®

Art.No.1022061 - BABY NASAL ASPIRATOR

EN	BABY NASAL ASPIRATOR	1
ES	ASPIRADOR NASAL PARA NIÑOS	2
IT	ASPIRATORE NASALE DEL BAMBINO	3
FR	ASPIRATEUR NASAL BÉBÉ	4
BG	БЕБЕШКИ АСПИРАТОР ЗА НОС	5
GR	ΡΙΝΙΚΟΣ ΑΝΑΡΡΟΦΗΤΗΡΑΣ ΜΩΡΩΝ	6
AR	شفاف أنف للأطفال	7
DE	NASENSAUGER FÜR BABYS	8
CZ	DĚTSKÁ NOSNÍ ODSÁVAČKA	9
HU	BABA ORRSZÍVÓ	10
RU	ДЕТСКИЙ АСПИРАТОР ДЛЯ НОСА	11
SRB/ME HR/BiH	BEBI ASPIRATOR ZA NOS	12
RO	ASPIRATOR NAZAL PT BEBELUȘI	13
NL	BABY NEUSASPIRATOR	14
AL	ASPIRUTOR PËR HUNDA E BEBES	15
TR	BEBEK BURUN ASPIRATÖRÜ	16
PL	ASPIRATOR DO NOSA DLA NIEMOWŁĄT	17
MK	БЕБЕ АСПИРУТОР ЗА НОС	18

Baby Care

find us on
www.lorelli.eu

BABY NASAL ASPIRATOR

PLEASE, READ THE INSTRUCTIONS CAREFULLY AND RETAIN THE PACKAGING FOR FUTURE REFERENCE.

For your child's safety and health:

WARNING!

- This product is not a toy.
- Never leave a child unattended while using this product.
- This product should only be used by an adult.
- Always use this product with adult supervision.
- Keep all components not in use out of children's reach.
- Keep out of reach of children.
- Throw away if there are any signs of damage or wear.

Care and Use

- Before use, wash your hands.
- Place the "mouth piece pipe" in your mouth and gently place the tip end of the nasal aspirator into one of your baby's nostrils.
- Gently suck from the "mouth piece pipe" without removing the tip end of the nasal aspirator from the baby's nostrils. The mucus is drawn up and collected into the aspirator.
- After collecting mucus, open up the tip and carefully dispose off the mucus hygienically from the aspirator.

To Clean

- Before first use, wash all parts of the nasal aspirator with warm water and a mild detergent, rinse thoroughly with clean water and dry.
- Always wash before and after each use.
- Dispose off the mucus, hygienically from the aspirator and wash under running tap water.
- Then make a mixture of warm water and a mild detergent. Open the nose tip and place all the parts of the nasal aspirator into the mixture. Wash thoroughly and repeatedly in this mixture of warm water and mild detergent until the nasal aspirator is clean. Rinse thoroughly with clean water and dry.
- Do not put in boiling water as this can damage the product.
- Do not use abrasive pads or cleaners.
- Do not clean or allow to come into contact with solvents or harsh chemicals. This could result in damage.
- Do not leave the product in direct sunlight or near heat because this can lead to damage.

Features

- peaceful sleep; soft and safe; effective; suitable for dual age group; easy and fast; silicone tip 0-3 month; without silicone tip 3+ month;

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Didis Ltd, Bulgaria, 9700 Shumen, 6 Trakia-iztok Street,